# REVISON OPTICS

<div><span><span></span></span></div>	<ul style="list-style-type: none"><li>Exposure Time: 3 minutes</li> <li>Drying Time: 20 minutes</li> <li>Pulses: Four (4)</li> <li>Cycle Type: Dynamic air-removal (Prevacuum)</li> <li>Temperature Set-Point: 135°C (275°F)</li> <li>Exposure Time: 3 minutes</li> <li>Drying Time: 20 minutes</li> <li>Pulses: Four (4)</li></ul>
---------------------------------------	---

**INLAY INSERTER CHUCK HANDLE**  
**INSTRUCTIONS FOR USE**  
**DEVICE: INSERTER HANDLE**  
 REF 607-0001

## INSTRUCTIONS FOR USE

The Inlay Inserter Chuck Handle is used in conjunction with the Inlay Inserter to extend the length of the device for ease of use while delivering the RVO corneal plug.


## PLEASE READ BEFORE USE

- Warnings**
  - The Inserter Handle is distributed NON-STERILE. Prior to use, it is the responsibility of the user to thoroughly clean and sterilize the instrument according to the recommendation of this document.
  - The Inserter Handle is intended to be used only for the purpose for which it was designed, and it is the responsibility of the physician to become familiar with the proper use of this instrument.
  - Wear appropriate protective equipment (gloves, eye protection, etc.) when reprocessing any medical device.
  - Do not use excessive force when cleaning and/or low pH (below 5) or high pH (above 10), which will damage the instrument's metal surface.

**Limitations on Processing**  
Proper repeated processing has a minimal effect on the instrument. End of life is normally determined by wear and damage due to use (see Inspection and Testing section below).

### Directions for Use

- The Inserter Handle has three components: the handle, the sleeve, and the chuck.

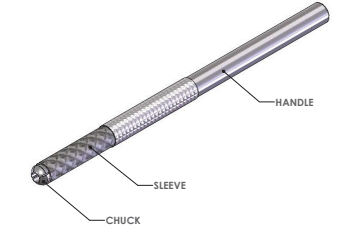- The Inserter Handle should be thoroughly cleaned and sterilized while disassembled below.
- To assemble the Inserter Handle, first insert the chuck's screw end through the sleeve via the ring end of the sleeve and then screw the chuck into the handle while holding the chuck round opening and the sleeve together.

- First, slightly loosen the chuck from the handle by holding the chuck's round opening end and the sleeve together.
- Twist the handle counter-clockwise a few times, so that the jaws of the chuck open enough for the shaft of the Inlay Inserter to fit in.
- Gently slide the shaft of the Inlay Inserter into the opening of the chuck.
- While holding onto the Inlay Inserter, the chuck and the sleeve all together, twist the handle clockwise to secure the Inlay Inserter in place.
- Verify Inlay Inserter and Inlay Inserter Handle are securely attached prior to use.

## CLEANING INSTRUCTIONS FOR FIRST USE AND REPROCESSING

- Preparation for Decontamination for First Use**
  - Carefully remove the Inserter Handle from its packaging.
  - Visually inspect the instrument to ensure there is no visible damage. Do not use the instrument if it is damaged.
  - Proprietary to thoroughly clean and sterilize the instrument as described in this document.

### Preparation at Point of Use

- Reprocess the instrument as soon as reasonably practical following use – up to 24 hours.
- As soon as possible, rinse the assembled instrument under cold running tap water (<27°C) for at least one (1) minute to remove any gross debris.

### Preparation for Decontamination

Disassemble (uncrew) the instrument into the individual components: chuck, sleeve and handle.

### Cleaning: Manual with Ultrasonic Bath

- The individual components of the instrument should be thoroughly cleaned by the following manual cleaning procedure:
  - Ensure that the ultrasonic bath is clean before use. Fill the bath to operation level with deionized or distilled water.
  - Use a solid tray or tote to contain the components and cleaning solution. Do not place the components directly into the ultrasonic bath.
  - Rinse each individual component under cold running tap water (<27°C) for at least one (1) minute each to remove major debris. Hold the sleeve vertically and rotate all components to expose all surfaces and cavity to flowing water.
  - Pat the components' surfaces with a soft, low lint cloth to remove excess moisture and debris.
  - Place all individual components in the solid tray or tote and add freshly prepared Prokren™ NPH High Performance Neutral Detergent cleaning solution, prepared according to the directions of the solution manufacturer, ensuring that all components are totally submerged in the solution.

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- Pat the components' surfaces with a soft, low lint cloth to remove excess moisture and debris.
- Place all individual components in the solid tray or tote and add freshly prepared Prokren™ NPH High Performance Neutral Detergent cleaning solution, prepared according to the directions of the solution manufacturer, ensuring that all components are totally submerged in the solution.
- Rinse each individual component under cold running tap water (<27°C) for at least one (1) minute each to remove major debris. Hold the sleeve vertically and rotate all components to expose all surfaces and cavity to flowing water.
- Final rinse individual components by pouring deionized water on them for a minimum of 30 seconds each.

**Drying**  
Pat-dry all components with a soft, low lint cloth.

### Inspection and Testing:

- Under magnification in a well-lit area, visually verify that all the components are clean and dry. If soil is still visible, repeat cleaning. If moisture is still visible, repeat the drying with the low lint cloth.
- Assemble the instrument's components by first inserting the chuck's screw end through the sleeve via the ring end of the sleeve and then screw the chuck into the handle while holding the chuck round opening and the sleeve together. It is possible that the components are still soiled. Repeat cleaning.
- If after repeat cleanings, the components still do not fit smoothly together, it is an indication that the threads are damaged; the handle cannot be used safely for its intended use.
- If the components fit smoothly together, repeat tightening and loosening the handle and chuck to observe to ensure that the chuck's jaws slightly open and close uniformly. If the jaws do not open when loosened, then the handle is damaged and cannot be used.
- If an instrument can be assembled properly and has no visible damage, disassemble (uncrew) it into individual components and move to the next step.

### Packaging

For steam sterilization: package the individual components of the Inserter Handle together in double pouches of a legally marketed sterilization pouch that is approved for the steam sterilization process.

### Sterilization: Moist Heat

- Cycle Type: Dynamic air-removal (Prevacuum)
- Temperature Set-Point: 132°C (270°F)
- Exposure Time: 4 minutes
- Drying Time: 20 minutes
- Pulses: Four (4)
- or
- Cycle Type: Dynamic air-removal (Prevacuum)
- Temperature Set-Point: 134°C (273°F)

<div><span><span></span></span></div>	<ul style="list-style-type: none"><li>Exposure Time: 3 minutes</li> <li>Drying Time: 20 minutes</li> <li>Pulses: Four (4)</li> <li>Cycle Type: Dynamic air-removal (Prevacuum)</li> <li>Temperature Set-Point: 135°C (275°F)</li> <li>Exposure Time: 3 minutes</li> <li>Drying Time: 20 minutes</li> <li>Pulses: Four (4)</li></ul>
---------------------------------------	---

**INLAY INSERTER CHUCK HANDLE**  
**INSTRUCTIONS FOR USE**  
**DEVICE: INSERTER HANDLE**  
 REF 607-0001

**Storage**  
In between use, store the packaged instrument in a dry and clean environment.

- Additional Information**
  - When sterilizing multiple instruments in one cycle, ensure that the sterilizer's maximum load is not exceeded.
  - The instructions provided above have been validated by the manufacturer of the medical device as being CAPABLE of preparing a medical device for re-use. It remains the responsibility of the processor to ensure that the reprocessing is actually performed using equipment, materials and personnel in the reprocessing facility achieve the desired results. This normally requires validation and routine monitoring of the process.

### SYMBOLS ON PACKAGING

SYMBOLS	EXPLANATION (English)
<span><span></span></span>	Manufacturer
<span><span></span></span>	Authorized Representative in the European Community
<span><span></span></span>	Batch Code
<span><span></span></span>	Date of Manufacture
<span><span></span></span>	Catalogue Number
<span><span></span></span>	Consult instructions for use
<span><span></span></span>	Non Sterile

**ReVision Optics, Inc.**  
25651 Atlantic Ocean Drive, Suite A1  
Lake Forest, CA 92630 USA  
Tel.: +1 949.707.2740  
Fax: +1 949.707.2744  
info@revisionoptics.com  
www.revisionoptics.com

**EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

RVO® is a registered trademark of ReVision Optics, Inc. Patent. ©2017 ReVision Optics, Inc. All rights reserved. Manufactured in the United States of America. 304-0040 Rev 5

RVO® is a registered trademark of ReVision Optics, Inc. Patent. ©2017 ReVision Optics, Inc. All rights reserved. Manufactured in the United States of America. 304-0040 Rev 5

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